

# Nova™



Fig. 1



Fig. 2



Fig. 3



Fig. 4

## To Set-up the Table:

- Place the folded table sitting on the rubber bumpers; with the heel of your hand, bump the UniLock™ tab between the face rest outlets and the table will open. (Fig. 1)
- Open the table up until the legs are fully extended. Remove the face rest. (Fig. 2)

## To Set the Table Upright:

- When the table is fully opened, tilt the table upright until all four feet are on the floor. Lift one end of the table, press down on the middle and tension the cables a little. (Fig. 3)

## To Change the Table Height:

- Choose the hole that represents the desired height and after removing the knob, put the bolt through that hole. Replace and tighten the knob by hand until snug. Repeat on all other leg extensions, setting all extensions at same height. (Fig.4)

NOTE: OAKWORKS® is not liable for injuries or damages sustained when table is not used in the manner prescribed.



Fig. 5



Fig. 6



Shiatsu Cable



Fig. 7



Fig. 8

## To Close the Table:

- Turn the table on its side with the rubber bumpers down. If you own a face-rest, store it under the table (Fig. 5). Partially close the table top.
- Fold the legs into the table and make sure the armrest is laying flat and the cables are inside of the table. Push the two halves of your table together to lock. (Fig. 6).

## When Using Shiatsu Option:

- With the table on its side, remove the wing nut from the bolt and remove long brace, cable and washers from the short brace. Reassemble them without the cable. Repeat on the second wing nut.
- Turn the UniLock™ clasp to the side (Fig. 7)
- Fold the legs against the underside and lay the table down flat on the floor. (Fig. 8).
- ***Shiatsu Cables must be installed as shown when using table in normal height ranges.***

MassageKing.com  
110 Brush Road,  
Broussard, LA 70518  
  
800-290-3932

## CAUTION

Dynamic Load Rating: 550 lbs.

Shiatsu option must be  
installed as shown.

Made in the U.S.A.  
with U.S. & Imported Parts

Inspected By: \_\_\_\_\_

Oakworks®  
Original

All portable tables: U.S. Patent 5,009,170  
SlideLock™: U.S. Patent 5,943,965  
Unilock™ Patent Pending

Part No. MMINWC0001 Nova Instructions Rev. 03.20.08

CE

OAKWORKS®, Inc.

923 East Wellspring Rd., New Freedom, PA 17349

Toll Free Phone: 800-558-8850  
Phone: 717-235-6807 FAX: 717-235-6798  
www.oakworks.com

OAKWORKS®  
Bodies Talk. We Listen.